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GB Safety instructions

Read the instruction manual carefully and use this device especially the safety instructions, and keep the instruction manual for future use.

Should you give this device to another person, it is vital that they also read the instruction manual for future use.

Deleting the memory

Press and hold the M button for approximately 3 seconds and all measured data stored will be deleted automatically after three beeps.

Blood pressure classification according to the WHO

These values have been specified by the World Health Organisation (WHO) without making any allowance for age.

Low blood pressure
systolic <100 diastolic <60

Normal blood pressure (green display area)
systolic 100 - 139 diastolic 60 - 89

Forms of hypertension
Mild hypertension (yellow display area)
systolic 140 - 159 diastolic 90 - 99
Moderate hypertension (orange display area)
systolic 160 - 179 diastolic 100 - 109
Severe hypertension (red display area)
systolic >180 diastolic >110

! WARNING

Blood pressure that is too low represents just as great a health risk as blood pressure that is too high. Fits of dizziness may lead to dangerous situations arising (e.g. on stairs or in traffic).

Error messages and error remedying

1. : Replace both batteries.
2. Er 1 (Error in measuring the systolic pressure)
3. Er 2 (Error in measuring the diastolic pressure) : Do not move.
4. Er 3 (Pressure system is blocked or sleeve is too tight)
5. Er 4 (Pressure system has a leak or sleeve is too loose during inflation)
6. Er 5 (Sleeve pressure above 300 mmHg)
7. Er 6 (Longer than 3 min. with sleeve pressure under 5 mmHg)
8. Er 7 (EPPROM access error)
9. Er 8 (Error when testing the functions on the device)
10. Er 9 (Error when testing the functions on the device)

Unit and LCD display

- 1 Battery Compartment
- 2 Blood Pressure Indicator
- 3 LCD screen (display)
- 4 M button (Memory Recall)
- 5 Wrist cuff
- 6 Save symbol
- 7 Display of Diastolic Pressure
- 8 Arhythmia display
- 9 Blood Pressure Indicator (Bar display)
- 10 Change Battery symbol

Scope of supply

Please check first of all that the device is complete. The following parts are included in the scope of supply:

- 1 MEDISANA blood pressure measuring device HGH
- 2 batteries (type AAA, LR03) 1.5V
- 1 storage box
- 1 instruction manual

Starting up

Insert / replace batteries

You should not insert the batteries before you can use your unit. The lid of the battery compartment is located on the left hand side of the unit. Open it, remove it and insert 2 AAA (type 1.5 V) batteries supplied. Ensure correct polarity when inserting the batteries (as marked inside the battery compartment). Close the battery compartment. Replace the symbols of the change battery symbol appears on the display or if nothing appears on the display after the unit has been switched on.

Guidelines / Standards

This blood pressure monitor meets the requirements of the EU standard for non-invasive blood pressure monitors. It is certified in accordance with EC Guidelines and carries the CE symbol (conformity symbol) „CE 0297“. The blood pressure monitor conforms to European standards EN 60601-1, EN 60601-1-2, EN 1060, EN 14971 and EN 980. The specifications of EU Guideline „93/42/EEC of the Council Directive dated 14 June 1993 concerning medical devices“ are met.

Electromagnetic compatibility:
The device complies with the EN 60601-1-2 standard for electromagnetic compatibility. Inquire at MEDISANA for details on this measurement data.

Technical specifications

Name and model: MEDISANA Blood pressure measuring device HGH
Display system: Digital display
Memory slots: 3 Vw with date and time
Measuring method: Oscillometric
Voltage supply: 3Vw, 2x1.5V battery AAA LR03
Measuring range blood pressure: 40 - 300 mmHg
Measuring range pulse: 40 - 180 beats/min.
Maximum measuring deviation of static pressure: ± 3 mmHg
Maximum measuring deviation of pulse values: ± 5 % of the value
Pump generation: Automatic with pump
Air release: Automatic
Autom. switch-off: After 1 min.
Operating conditions: +5 °C to +40 °C
Storage conditions: -80 % max. relative humidity
Dimensions: 83 x 64 x 28 mm
Cuff: 300 x 70 mm, circumference 14 - 19,5 cm, arm circumference 41 - 49,5 cm
Weight (approx.): 110 g without batteries
EAN number: 40 15588 51430 2

Fitting the cuff

Place the cuff on the left wrist with the palm of the hand facing upwards (fig. 1).

The distance between the cuff and the palm of the hand must be approx. 1 cm (fig. 2).

Fasten the cuff around your wrist using the Velcro fastener (fig. 3).

The correct position

- 1. Carry out the measurement whilst seated.
- 2. Relax your arm and place it comfortably on a table or surface.
- 3. Raise the wrist so that the cuff of the monitor is at heart level (fig. 4, position a = too high, b = correct, c = too low).
- 4. Keep still during the measurement; do not move and do not talk as this may alter the measurements.

Taking a blood pressure measurement

1. Switch the unit on by pressing the START button .
2. If the START button is pressed, a long beep is heard and all display characters are shown (display test). This test can be used to check that the display is indicating correctly and in full function. The test will only occur if the previous measurement. If no measurement has been taken, zeroes will appear next to the date and time.
3. The unit is ready for measurement, the number 0 is displayed. It automatically inflates the cuff slowly in 10 seconds. The display shows the pulse rate and the display changes continuously during the time it continues to inflate.
4. The unit inflates the cuff until sufficient pressure has built up for a measurement. Then the unit slowly releases the air from the cuff and carries out the measurement.
5. When the measurement is finished, the cuff is deflated completely. The systolic and the diastolic blood pressure, as well as the pulse rate, appear in the display in succession.
6. The display shows the pulse rate and the display changes continuously during the time it continues to inflate.
7. The unit inflates the cuff until sufficient pressure has built up for a measurement. Then the unit slowly releases the air from the cuff and carries out the measurement.
8. When the measurement is finished, the cuff is deflated completely. The systolic and the diastolic blood pressure, as well as the pulse rate, appear in the display in succession.
9. The display shows the pulse rate and the display changes continuously during the time it continues to inflate.
10. The unit inflates the cuff until sufficient pressure has built up for a measurement. Then the unit slowly releases the air from the cuff and carries out the measurement.
11. When the measurement is finished, the cuff is deflated completely. The systolic and the diastolic blood pressure, as well as the pulse rate, appear in the display in succession.
12. The display shows the pulse rate and the display changes continuously during the time it continues to inflate.

! WARNING

Do not take any therapeutic measures on the basis of a self-measurement. Never alter the dose of a medicine prescribed by a doctor.

The unit switches off automatically after a minute if no buttons are pressed. The unit may also be switched off using the START button .

Displays of stored values

You can store up to 60 measurements (blood pressure and pulse with date and time) in the memory of your blood pressure monitor. The measured values are stored automatically.

! WARNING

Do not take any therapeutic measures on the basis of a self-measurement. Never alter the dose of a medicine prescribed by a doctor.

The unit switches off automatically after a minute if no buttons are pressed. The unit may also be switched off using the START button .

Displays of stored values

You can store up to 60 measurements (blood pressure and pulse with date and time) in the memory of your blood pressure monitor. The measured values are stored automatically.

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