99.99 mm

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
Digital display breath alcohol Ritmix RAT-310

User manual

**ATTENTION!**  
Before the using the alcohol tester, please read carefully the user manual first and follow its recommendations

Usage

Alcohol tester Ritmix RAT-310 is for making the personal test of measuring the alcohol in the exhaled vapor. Tester recalculates the data and provides the results as concentration of alcohol in blood.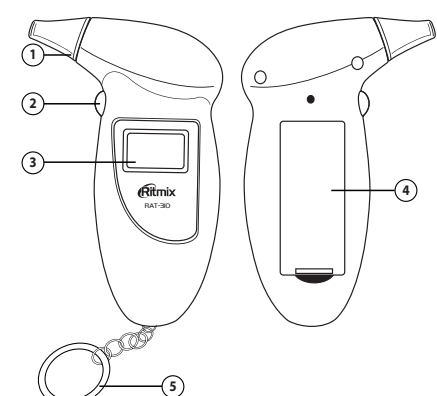

- (1) Breath inhaler
- (2) Power button
- (3) LCD display
- (4) Battery compartment
- (5) Key chain


**NOTE**  
**BAC – Blood -Alcohol Concentration, 0.05% BAC:** shows that there is 0.05 g of alcohol in blood in each 100 ml of blood.

Features

- 1. Digital alcohol test with audible alert
- 2. Range: 0.00‰-1.9‰ (0.0%-0.19% BAC or 0.0-1.9 g/l BRAC)\*
- 3. Quick response and resume
- 4. Auto power off
- 5. Use mouthpiece
- 6. Key chain
- 7. Use 2 x "AAA" alkaline batteries are not included

Installing battery

- 1. Slide open the battery compartment lid from the back of the unit.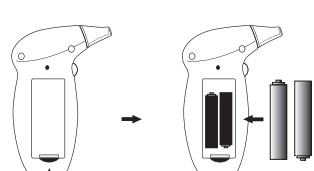


- 2. Insert two "AAA" alkaline batteries with correct polarity as indicated.
- 3. Replace the battery compartment lid. Use only alkaline batteries. Change all the batteries at once, don't use the batteries which were used before. During the long period of storage please insert the batteries.

How to use the alcohol tester

Warm up

- 1. Press and hold the power button for 1 second, the alcohol tester LCD will turn on with one beep sound.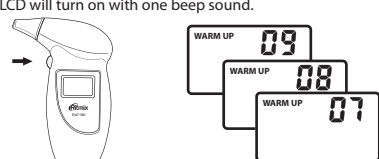


- 2. The "warm up" symbol is displayed, and the 10 to 00 countdown is started that indicate the tester is in warm-up mode.

Note

It always has a large number of gas & impurity adsorbed on the surface of alcohol tester, when first time taking out the tester from the packaging, or after long time no using for the tester, it need more time for cleaning and warm up the sensor, you should repress the power button and retry several times if possible.

Test

- 1. The "Blow" symbol is displayed when warm-up is succeeding, and the 10 to 00 countdown is started that indicate the tester is in testing mode. After finishing of the warm-up process the display will show "Blow".

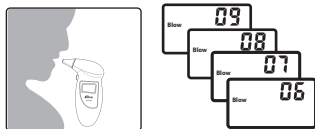- 2. Blow into the breath inhaler for 10 seconds with normal power after a deep inhalation, till you don't hear the sound signal.
- 3. Read the test result from the alcohol tester LCD.
- 4. The "Caution" symbol is displayed if the alcohol concentration is the level: 0.2‰-0.5‰ (0.02% BAC - 0.05% BAC or 0.2 g/l - 0.5g/l BRAC)
- More than 0.5‰ (0.05% BAC or 0.5g/l)

- 5. The "Danger" symbol is displayed with beep audible alert if the alcohol concentration is the level: Over & equal 0.05% BAC (or 0.5g/l) (BAC: Blood Alcohol Concentration, 0.05% BAC = 0.05g/100mL BAC = 0.5g/L BAC)
- 6. If no blow into the breath inhaler at the test mode, the LCD will display "0.00 %BAC & 0.0g/l" value.

Note

To obtain more accurate result, manufacturer recommends you to check repeatedly 2 or 3 times. Don't use the alcohol tester if there is a strong wind or there is dust. If the cigarette smoke or saliva will fall on tester, it might cause the damage of the alcohol sensor. Keep the alcohol tester in a dry and clean place at a temperature +5...+40°C. Please do not dismantle and modify this device. Please use this device according to the user manual.

Troubleshooting

If the alcohol tester provides the wrong results, than please blow the sensor with the dry warm air for few minutes.

Auto power off

After the result is display for about 10 seconds, the "OFF" symbol will appear for 2-3 seconds and the alcohol tester be turned off.

Replace the mouth

- 1. Turn the mouthpiece counterclockwise for its separation.
- 2. Place the new mouthpiece and turn it clockwise.

Replace battery

The LCD flickering indicates low power of batteries; please replace two "AAA" alkaline batteries.

Specifications

- \* Sensor: advanced semiconductor oxide alcohol sensor
- \* Range: 0.00%-0.19% BAC % (0.0-1.9g/l)
- \* Accuracy: 0.01% BAC (0.1 g/l)
- \* Power input: 2x1.5V "AAA" alkaline battery
- \* Operating time: warm up: 10s, response: 10s
- \* Auto power off: 1 minute approx
- \* Dimension: 118x70x28mm
- \* Environmental temperature: +5°С...+40°С

Precaution

After drinking, it is recommended to last after 20 minutes, it is because 20 minutes needed approximately for alcohol to be absorbed into blood from the digestive organs, and residual alcohol remaining in the mouth takes this long to dissipate. For accurate result, do not repeat next testing within 3 minutes afterlast time.

Do not smoke and eat for 20 minutes prior testing. Do not use chemicals to clean the tester directly. For cleaning the mouth piece please use the dry material or paper napkin. Do not blowing liquid into the breath pipe. Do not use any excitant substances near the tester, such as paint, insecticides, alcohol. Do not place the tester in the closed environment with contaminated air. Each individual's body responds differently to alcohol. Drink one glass of wine somebody can feel slightly drunken, somebody can feel heavily drunken. Diabetes or low-calorie diet or other factors are leading to the rising of the concentration of ketones in exhalation and can cause false positive results. Any of Alcohol testers doesn't give the 100% of guarantee of the accurate results. The most accurate results give blood measuring test. That's why if the road patrol has stopped you, and alcohol tester showed the presence of alcohol in blood, require the blood test. Reports of Worlds Health Organization shows that if the alcohol concentration reaches the 0.08 g/100 ml, than the risk of road accidents increases in 2.5 times to compare if there is no alcohol in blood. If the alcohol concentration in blood reaches 0.1 g/100 ml, risk of road accidents increases in 4.7 times, than there is no alcohol in blood. Even if the slight quantity of alcohol is drunken it increases risk of road accidents in 2 times. The unit is designed to be used as an aid to promote reasonable driving. It should not be used in anticipation of the use or operation of vehicles, machinery, or equipment. The unit makes no references whatsoever to blood alcohol content or other such measures of intoxication promulgated by government, medical or other agencies or bodies. The result of this alcohol gives BAC for reference only; manufacturer does not to take any legal responsibility. Test results of alcohol tester are not probative. Don't drive in a drunk condition of any degree of intoxication.

Warranty

The company-producer guarantees the normal work of device during the all warranty period if the device will be used according to the user manual. Warranty period for this device is 6 months since the day of the selling through the commercial network. Time of service for this device is not less than 1 year. The company-producer doesn't accept reclamations in case of the mechanical damage of device, in case of dismantle, wrong usage, or using not in a proper way. This device will serve you for many years if careful handling.

Company «Ritmix» has the right to change the Specification without notification.

\*-information about range 0.00 – 1.9 g/l BRAC is given only as referential information and it is not shown on the display of the alcohol tester

99.99 mm